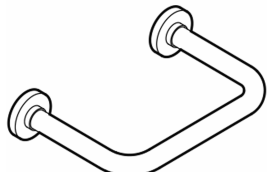
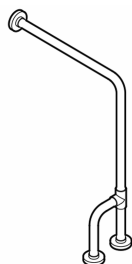
G01JAS01  
G01JAS02  
G01JAS03  
G01JAS04  
G01JAS05  
G01JAS06  
G01JAS07  
G01JAS41  
G01JAS42  
G01JAS43  
G01JAS44

G01JAS15

G01JCS38  
G01JCS39

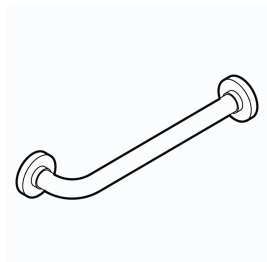
G01JAS08  
G01JAS20


G01JER07  
G01JEL07


G01JOS01

G01JAS32  
G01JAS33  
G01JAS34

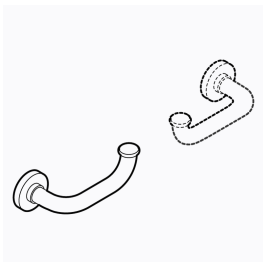G01JES10

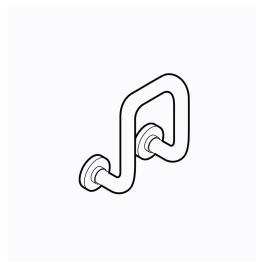G01JAS21

**IT**

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**EN**

**INSTRUCTIONS AND  
PRECAUTIONS FOR SAFE  
USE.**

Prior to installing and using the device, carefully read the instructions for use and – in particular – the safety precautions, and comply with them. Store this manual for reference for the entire life-cycle of the device. Should this manual get damaged, or if certain sections are difficult to understand or give rise to doubts, prior to using the device contact the company at the address indicated on the last page. Any changes to the information provided in the instruction manual – if deemed important for the user's safety – shall be announced through the Ponte Giulio Internet website and

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notified to all authorised dealers.

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#### GENERAL INFORMATION

The device is safe to use, even for people with reduced visual, reading or cognitive abilities. There are no circumstances hampering ease of use of the product.  
The device must not be dismantled and does not have dismantable parts.  
The device's overall weight does not exceed 10 kg.  
The device does not contain carcinogenic, mutagenic or toxic substances.  
It is capable of withstanding corrosion and temperature variations.

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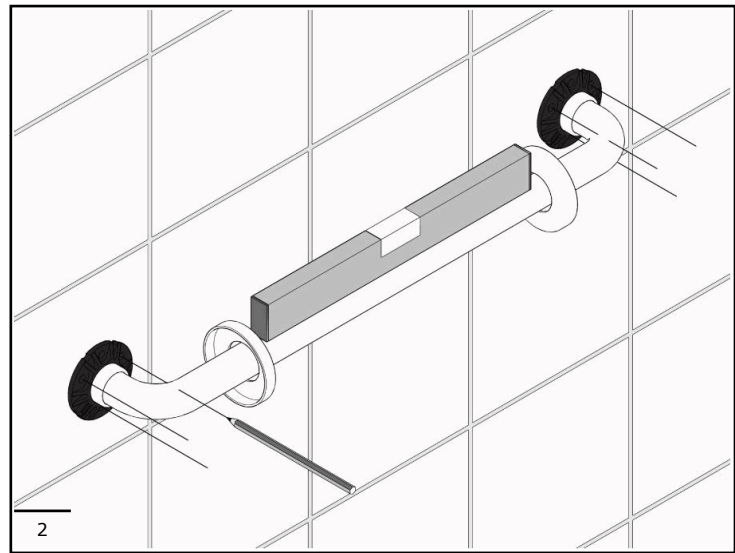
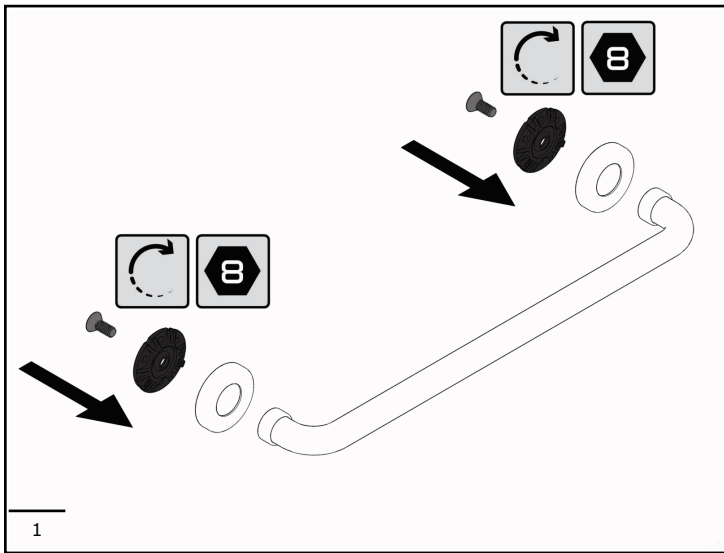
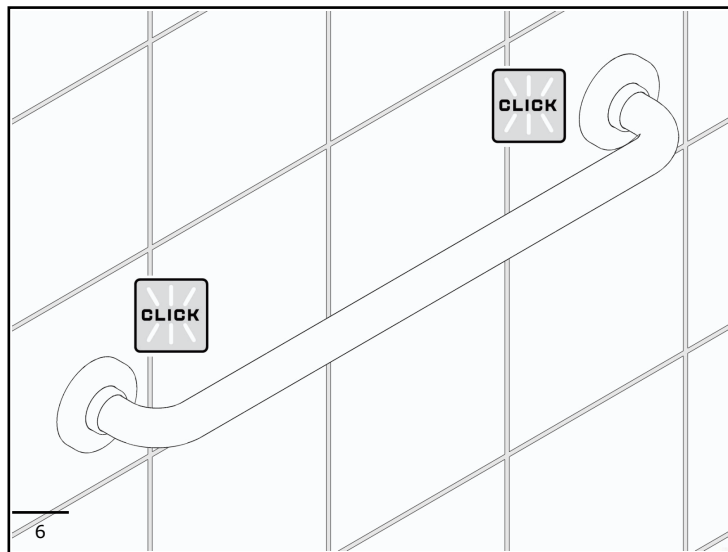
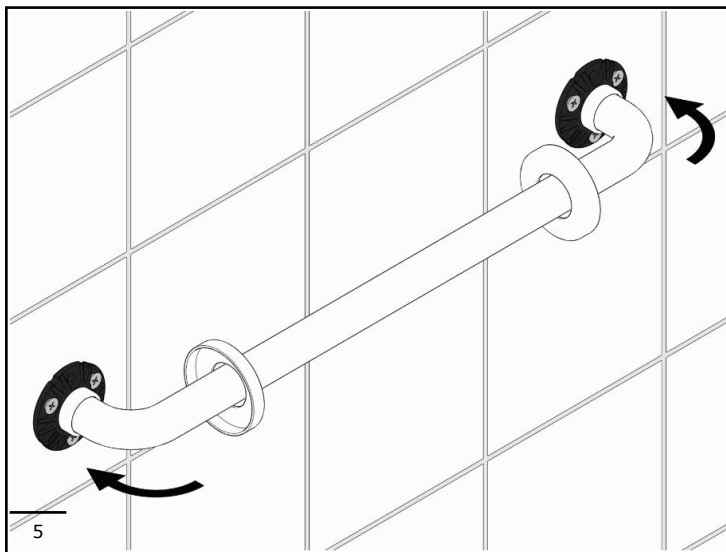
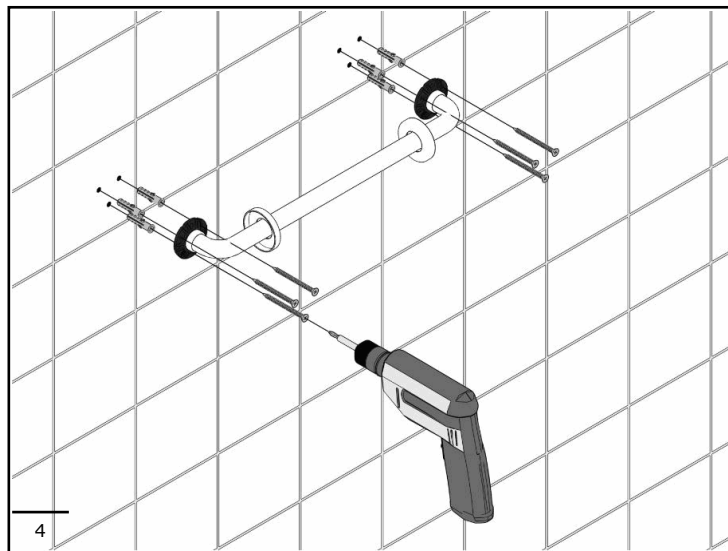
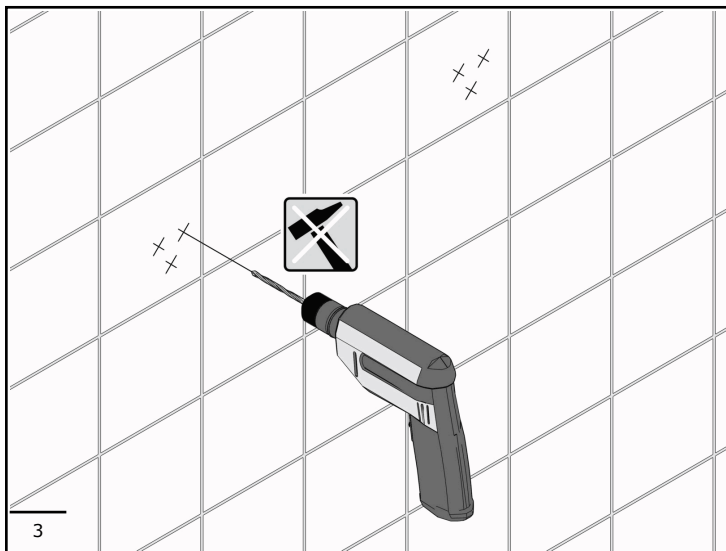
**EN**

**INSTRUCTIONS FOR USE**

The device is a grab bar to be used exclusively for supporting the user or resting on by gripping it with the hand. Using the device improperly, for example for gymnastics exercises, can cause injuries with serious consequences. Do not subject the device to loads exceeding for which it was designed. Exceeding this load can cause permanent damage to the device or harm to people. To ensure correct use of the device, the latter must be left free of any objects and be easily reachable.

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**EN****SAFETY PRECAUTIONS**

The device must be installed by trained and qualified personnel (bricklayer or professional plumber) who must verify whether the fixing elements, if provided, are suitable for the relevant wall.

To guarantee stability and safety, the device must be attached with wall plugs and screws suitable for the relevant type of wall.

Do not subject the device to loads exceeding those specified.

If the device should become unstable, suspend its use until it is properly checked by the relevant technician.

The manufacturer declines all responsibility for installation of the device not conforming in this manual for use and maintenance.

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**EN****CLEANING**

Clean the surface of the device using a damp cloth wetted with liquid neutral soap (e.g. dish soap). It is forbidden to use abrasive liquid detergents containing chlorine, strong acids or bases, concentrated solvents such as acetone and trichloroethylene or derivatives, which may cause the material too dull.

It is forbidden to use hot air sources for drying.

Disinfect the device's surface using a bleach-type product at least once a week.

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#### CONDITIONS OF USE

The device is not affected by the room's temperature, though it must NOT be installed in environments subject to temperatures below 4°C or above 40°C.

Do not install in saunas. Not comply with the indicated temperatures may damage the plastic coating of the device.

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#### DISPOSAL

The user is responsible for the device's disposal, which must occur in accordance with the laws of the country in which the device is installed. The device is not manufactured using substances that are hazardous to human health and the environment, and all parts comprising it are recyclable and/or reusable. In order to safeguard the environment, the device also contains a certain amount of recycled materials.

Keep away from children packaging; risk of suffocation.

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#### FURTHER INFORMATION

For further information, please visit our website or contact the our dealer zone

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#### NOTE

Ponte Giulio reserves the right to make changes to the products or to their specifications without prior approval or notification to the customer for:  
- Changes required for safety purposes,  
- Changes that do not adversely affect the product performance,  
- Changes needed to meet the

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product specifications,  
- Changes needed to comply with legal requirements or applicable regulations

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